Ce produit est conforme avec la directive 93/42/EEC des dispositifs médicaux.
 This product is conform with the directive 93/42/EEC for medical products.
 Dit product beantwoordt aan Richtlijn 93/42/EEC over medische apparatuur.
 Este producto es conforme con la Directiva 93/42/EEC relativa a los productos médicos.
 Questo prodotto è in conformità con la Direttiva 93/42/EEC relativa ai dispositivi medicali.
 Este produto está conforme à Directiva 93/42/EEC relativa aos dispositivos médicos
 Dieses Erzeugnis entspricht der Richtlinie 93/42/EEC über Medizinprodukte.
 Соответствует требованиям директивы 93/42 ЕС о медицинских изделиях.

Référence G / Reference G / Referentie G / Referencia G / Codice G / Referência G / Referenz G / Код G :	491301	491601
	492351	
Référence P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P / Код P :	049130 - 049140 - 049150	049160 - 049180 - 049220
	049230 - 049270 - 049330	049360

Fonction	Permet à l'utilisateur de prendre appui en toute sécurité.
Function	Allows the user to lean safely.
Functie	Zo kan de gebruiker in alle veiligheid ondersteund worden.
Función	Permite al usuario apoyarse con seguridad.
Funzione	Permette di prendere appoggio con sicurezza.
Função	Permite ao utilizador apoiar-se em toda a segurança.
Funktion	Zum sicheren Abstützen des Benutzers.
Назначение	Служит для пользователя надежной опорой.

Descriptif Description Beschrijving Descripción Descrizione Descrição Beschreibung Описание			049130 - 491301	150 kg
			049140 049150 049160 - 491601 049220 - 492351 049230 049270 049360	
			049180 049330	120 kg
			049130 - 491301	0,500 kg
			049140	0,585 kg
			049150	0,655 kg
			049160 - 491601	0,735 kg
			049180	0,880 kg
			049220 - 492351	0,480 kg
			049230	0,880 kg
			049270	1,000 kg
			049330	1,350 kg
			049360	2,200 kg

Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции	A49130137
Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания	22/03/2011

Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакции	05
Date / Date / Datum / Fecha / Data / Data / Datum / Дата	01/02/2016

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Do not install this equipment before reading the following instructions: **Fitting instructions.** Advised height 80 cm from the ground. Straight grab bars can be installed obliquely (135°) or horizontally. Before drilling into the wall, physically hold the product against the wall and mark the position of the holes to be drilled. Plugs have to be selected according to the type of wall. The fixings points have to be necessarily respected. **Care and maintenance. IMPORTANT:** In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water. **Safety warnings.** Grab bars add security to the user if they are fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. **DO NOT** use grab bar as a step to reach area, do not sit or hang on grab bar; use it for assistance **ONLY. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this grab bar.** At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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Fabricants : PELLET ASC et GODONNIER SAS
Distributeurs : GODONNIER SAS et PELLET ASC