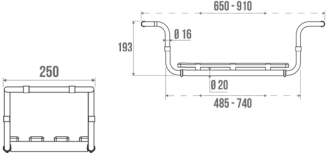
Poids maximum utilisateur / Max. user weight / Maximumgewicht gebruiker / Peso máximo utilizador / Peso max. Utente/Peso máximo do utilizador / Maximales Benutzergewicht / Макс. вес пользователя

Poids net / Net weight / Nettogewicht / Peso neto / Peso netto / Peso líquido / Nettogewicht / Вес нетто

<b>Référence G / Reference G / Referentie G / Referencia G / Codice G / Referència G / Referenz G / Код G :</b>		133000
<b>Fonction</b> <b>Function</b> <b>Funcție</b> <b>Función</b> <b>Funzione</b> <b>Função</b> <b>Funktion</b> <b>Назначение</b>	Permet de prendre un bain en position assise. Allows to take a bath in a seated position. Zo kan u zittend een bad nemen. Permite tomar un baño en posición sentada. Permette di prendere un bagno in posizione seduta. Permite tomar um banho sentado. Zum Baden in Sitzposition Предназначено для облегчения принятия ванны в положении сидя.	
<b>Descriptif</b> <b>Description</b> <b>Beschrijving</b> <b>Descrizione</b> <b>Descrição</b> <b>Beschreibung</b> <b>Описание</b>		 90 kg  3,023 kg
<b>Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции</b>		500001097
<b>Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания</b>		20/10/2002
<b>Révision n / Revision No. / Revisie nr. / Revisión n° / Revisione n / Revisão n° / Revisionsnr. / № редакции</b>		08
<b>Date / Date / Datum / Fecha / Data / Data / Datum / Дата</b>		26/01/2022

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 **Do not install this equipment before reading the following instructions:**  **Fitting instructions.** Proper fitting of the supports against the sides

of the bathtub is essential for your safety. Once you have obtained the desired width by sliding the tubes, tighten the thumbwheels moderately to avoid damaging the thread. When using as a stool, ensure the legs are pushed fully into the closed position. There may be fine scratches on the moving parts of the tubing. These are caused during testing at our factory and do not in any way alter the appearance or the working of the product once set up.

 **Care and maintenance.** IMPORTANT: In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. The disinfection has to be made according to the recommendations of use of the various products of disinfection. After use of a cleaning product, make a rinsing in the clear water.  **Safety warnings.** The bath seat adds security to the user if it is fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. The bath seat is designed to be used in seated position. To guarantee the user's safety, it is not advised to rise up on the seat. To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this bath seat.  At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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