<b>Référence G / Reference G / Referentie G / Código G / Codice G /</b> <b>Referência G / Referenz G :</b>		488501
<b>Référence P / Reference P / Referentie P / Código P / Codice P /</b> <b>Referência P / Referenz P :</b>		047635
<b>Fonction</b> <b>Function</b> <b>Functie</b> <b>Función</b> <b>Funzione</b> <b>Função</b> <b>Funktion</b>	Permet de sécuriser la position assise sous la douche ou dans les toilettes. Enables to reassure the seating position on the shower or toilets. Zorgt voor een veiligere zitplaats onder de douche of op het toilet. Permite hacer más segura la posición sentada en la ducha o en el WC. Garantisce una maggiore sicurezza alla posizione seduta sotto la doccia o nei WC. Permite proteger a posição sedentária no duche ou na banheira. Schenkt Sicherheit beim Sitzen unter der Dusche oder auf der Toilette.	
<b>Descriptif</b> <b>Description</b> <b>Beschrijving</b> <b>Descrición</b> <b>Descrizione</b> <b>Descrição</b> <b>Beschreibung</b>		60 kg 1,100 kg
<b>Numéro notice / Instruction sheet no / Fichenummer / Número instrucciones / Numero manuale / Número do manual /</b> <b>Gebrauchsanweisungsnummer</b>		S00001631
<b>Date de création / Creation's date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum</b>		22/12/2006
<b>Révision n° / Revision no. / Revisie nr. / Revisión n° / Revisione n° /</b> <b>Revisão n° / Revisionsnr.</b>		07
<b>Date / Date / Datum / Fecha / Data / Data / Datum</b>		08/07/2015

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**GB Do not install this equipment before reading the following instructions:**

**Fitting instructions.** Necessary accessories: 6 plugs adapted to the wall-mounted support and 6 stainless steel screws. Important: Ø 4,5 mm with screw head Ø inferior to 7 mm. Before drilling the wall, physically hold the product against the wall and mark the position of the holes to be drilled. The 3 fixing points have to be necessarily respected. When the support has been fixed, fit the armrest with the support axis. Insert the provided screws through legs and the support axis, then screw. After, fit the superior protection cover first, and the inferior one, after. Assemble the 2 covers with screws. Recommended height: 70-75 cm from the ground. Max. authorized weight : 60 kg. **Care and maintenance.** **IMPORTANT:** In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water.

**Safety warnings.** Enables to reassure the seating position on the shower or toilets. Take care, do not use this product as a grab bar. The armrest adds security to the user if it is fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. To guarantee the user's safety, it is not advised to rise up on the armrest. **DO NOT USE** it as a grab bar. To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this armrest. At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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**Distributeur : GODONNIER SAS**